Manufacturer and Product Information

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.
For Technical Assistance call:
Diamond Diagnostics Technical Services at 1-508-429-0450

Contents:	1 – pO ₂ Electrode 1 – Cleaning Solution (RD-943905E) 1 – Cuvette
Intended Use:	Use with Radiometer ABL 5xx Series analyzers.
Storage:	Store the pO ₂ electrode at room temperature (18-25°C) until you are ready to use it in your system.
Precautions:	Following pO ₂ electrode replacement or membrane change, automatic calibrations are changed to the following Sequence: 1. First 4 hours, alternate in 30 minute intervals Gas Cal 1 and Gas Cal 2, ending with a Total Cal . 2. Next 20 hours, every hour Gas Cal 1, every second hour alternate with Gas Cal 2.

Procedure

- Installation:**
1. Remove the cover of the temperature controlled electrode unit.
 2. Press the pO₂ amplifier housing down, pull the metal tab outward, and lift the amplifier housing.
 3. Remove the old pO₂ electrode.
 4. Unpack the new pO₂ electrode.
 5. Add 5 drops of Electrode cleaner, RD-943905E, into the Cuvette provided with the electrode.
 6. Soak the electrode in the vial for 1 hour.
 7. Remove the electrode and rinse with plenty of distilled or deionized water.
 8. Gently shake excess water from the electrode.
 9. Obtain a box of pO₂ membranes in jackets. Remove the foil cover.
 10. Press the electrode firmly into the jacket.
 11. Remove the jacketed electrode from the box.
 12. Look for bubbles between the membrane and the electrode. If bubbles are observed, gently move the electrode up and down in the electrolyte to release the air bubbles. Press the electrode back into the jacket again.
 13. Using a dry clean cloth or paper towel, thoroughly wipe and dry the gold contacts of the electrode.
 14. Place the newly prepared electrode in the correct section of the measuring chamber.
 15. Gently press the amplifier housing down onto the electrode.
 16. Record in the maintenance schedule that the pO₂ electrode has been replaced.
 17. If all maintenance steps have been performed, place the cover over the temperature controlled electrode unit and press **Restart Analyzer** key below the display in scheduled maintenance menu.

Warranty

Warranty = 12 months: Warranty is calculated from date of shipment from Holliston, MA.

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Diamond Diagnostics Technical Services at 1-508-429-0450

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